

	< 60	60 to < 70	70 to < 80	≥ 80
20/10 or better	0 / 36 (0%)	0 / 128 (0%)	0 / 155 (0%)	0 / 47 (0%)
20/16 or better	6 / 36 (16.7%)	12 / 128 (9.4%)	4 / 155 (2.6%)	2 / 47 (4.3%)
20/20 or better	26 / 36 (72.2%)	85 / 128 (66.4%)	78 / 155 (50.3%)	22 / 47 (46.8%)
20/25 or better	33 / 36 (91.7%)	110 / 128 (85.9%)	121 / 155 (78.1%)	33 / 47 (70.2%)
20/30 or better	36 / 36 (100%)	125 / 128 (97.7%)	143 / 155 (92.3%)	41 / 47 (87.2%)
20/40 or better	36 / 36 (100%)	127 / 128 (99.2%)	152 / 155 (98.1%)	45 / 47 (95.7%)
20/50 or better	36 / 36 (100%)	127 / 128 (99.2%)	153 / 155 (98.7%)	47 / 47 (100%)
20/60 or better	36 / 36 (100%)	127 / 128 (99.2%)	154 / 155 (99.4%)	47 / 47 (100%)
20/80 or better	36 / 36 (100%)	127 / 128 (99.2%)	154 / 155 (99.4%)	47 / 47 (100%)
20/100 or better	36 / 36 (100%)	127 / 128 (99.2%)	155 / 155 (100%)	47 / 47 (100%)
20/200 or better	36 / 36 (100%)	127 / 128 (99.2%)	155 / 155 (100%)	47 / 47 (100%)
Worse than 20/200	0 / 36 (0%)	1 / 128 (0.8%)	0 / 155 (0%)	0 / 47 (0%)
Not Reported	0	0	0	0
Total	36	128	155	47



http://www.lenstec.com
ISO13485 Registered Company

WORLD HEADQUARTERS: LENSTEC, INC., 1765 Commerce Ave N, St. Petersburg, Florida 33716, U.S.A.

+1 (727) 571-2272 +1 (727) 571-1792 lenstec@lenstec.com

LENSTEC (BARBADOS) INC., Airport Commercial Centre, Pilgrim Road, Christ Church BB17092, Barbados

+1 (246) 420-6795 +1 (246) 420-6797 lenstecbarbados@lenstec.com

INSTRUCTIONS FOR USE
SOFTEC HD™, SOFTEC HDO™, SOFTEC I™ AND SOFTEC HDM™
POSTERIOR CHAMBER INTRAOCULAR LENSES (PCIOLs)



Cumulative Adverse Event through 1 year	Softec HD™ PCIOL Incidence	FDA PCIOL Grid n = 300
Cystoid Macular Edema	0.8%*	6.0%
Hypopyon	0%	1.8%
Endophthalmitis	0%	1.0%
Dislocated Lens (from Posterior Chamber)	0%	1.0%
Pupillary Block	0%	1.0%
Retinal Detachment	0%	1.8%
Secondary Surgical Intervention**	0.8%	2.6%

Persistent Adverse Event at 6 mths and/or 1 year	Softec HD™ PCIOL Incidence	FDA PCIOL Grid n = 300
Corneal Stromal Edema	0%	1.8%
Cystoid Macular Edema	0.8%*	2.2%
iritis	0.3%	1.8%
Raised IOP Requiring Treatment	0.3%	1.8%

*Identical cases reported in persistent & cumulative CME rows

**All unrelated to Softec HD™ PCIOL

ADVERSE EVENTS

Cumulative adverse events consist of all adverse events (AEs) that occurred at any point in postoperative follow-up during the first year after Softec HD™ PCIOL surgery.

Table 4 presents all cumulative adverse events through the 1 year visit (330-420 days); Table 5, all persistent adverse events at 6 months (120-180 days) and 1 year visits. The overall incidence of cumulative and persistent IOL Grid adverse events in the Softec HD™ PCIOL Study Group (n = 366) was 2.2% (CME 0.8%, secondary surgical interventions 0.8%, iritis 0.3% and raised IOP requiring treatment 0.3%).

Non-IOL Grid AEs included 9 haptic break AEs at the time of the initial surgery and 1 sub-retinal hemorrhage.

EXPIRATION DATE

The expiration date on the lens package is the sterility expiration date. Do not use the IOL after the expiration date.

RETURNS POLICY

Contact your Lenstec representative regarding the return goods policy. Return the lens with full identification and the reason for the return. Label the return package as a biohazard.

PATIENT REGISTRATION AND REPORTING

A Patient Identification Card is included in the package. This is to be completed and given to the patient, together with instructions to keep the card as a permanent record to be shown to any eye practitioner the patient consults in future. Self-adhesive lens identification labels are provided for use on the Patient Identification Card and other clinical records.

Adverse events/complaints that may reasonably be regarded as lens-related and that were not previously expected in nature, severity, or degree of incidence should be reported to Lenstec (Barbados) Inc., Airport Commercial, Pilgrim Road, Christ Church, Barbados; Tel: 246-420-6795 • Fax: 246-420-6797; Email: Feedback@lenstec.com or contact your Lenstec representative.

BIBLIOGRAPHY

- Holladay, J. Quality of Vision: essential optics for the cataract and refractive surgeon. Pgs 27-35
- Apple DJ, Arthur S, et al. Effect of heparin surface modification in reducing silicone oil adherence to various intraocular lenses. *Journal of Cataract & Refractive Surgery*. October 2001, Vol 27 No 10, 1662-1669.
- Hayashi H, Hayashi K, et al. Quantitative comparison of posterior capsule opacification after polymethylmethacrylate, silicone and soft acrylic intraocular lens implantation. *Arch Ophthalmol*. 1998; 116: 1579-82.
- Apple DJ, Federman JL, et al. Irreversible silicone oil adhesion to silicone intraocular lenses. A clinicopathologic analysis. *Ophthalmology*. June 1996. Vol 103, 1555-1561.
- Pandey SK, Apple DJ, et al. Posterior Capsule Opacification: A Review of the Aetiopathogenesis, Experimental and Clinical Studies and Factors for Prevention. *Indian J Ophthalmol* 2004; 52:99-112.
- Apple DJ, Kleinmann G, et al. A new classification of calcification of intraocular lenses. *Ophthalmology* Jan 2008, Volume 115, Issue 1, Pages 73-79.
- ISO 11979-7 Ophthalmic Implants – Intraocular lenses – Part 7: Clinical Investigations; 2006.



PI28 Rev 18



