

Silicone IOL Delivery System



1. Open the packaging and remove the Silicone Delivery System device.



2. Hold the device in one hand as shown.



3. Grasp the plunger knob with the other hand. Note, below, how the lens is flat within the base of the device.




4. Push the plunger all the way in. Note, below, that the lens is now folded and extends out beyond the device.



5. Now pull the plunger all the way out, to release the folded lens.



6. Using your implantation forceps, grasp the folded lens.

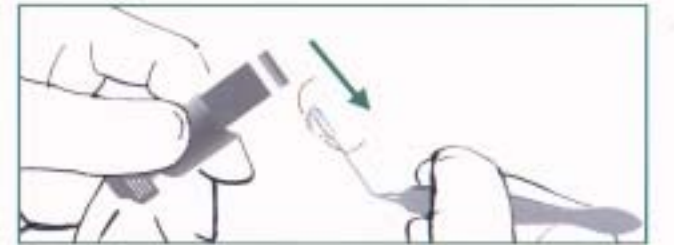

7. Slide the lens out of the Delivery System, as shown by the arrow. You are now ready for implantation.

To Reinsert the Lens into the Delivery System...

If the lens is not held securely by your forceps after removal from the Delivery System, you can reinsert the lens back into the device. Place the lens in an "S" configuration, as shown, depress the plunger slightly to hold the lens in place and repeat the procedure from Step 2.



For Lens Injector Users

Proceed through Step 5 and remove the lens with your forceps.