

# Acrylic IOL Delivery System

*Note: Lenstec recommends that the lens be implanted within 2 minutes after being removed from saline.*



1. Unscrew the cap...

2. And remove the Delivery System device with forceps...

### Lens Injector Users

Proceed through Step 4, then pull back on the plunger and remove the lens with your forceps.




3. Hold the device securely in one hand as shown...



4. And grasp the plunger knob with the other hand. Note, below, how the lens is flat within the base of the device.



5. Push the plunger all the way in. Note, below, that the lens is now folded and extends out beyond the device.

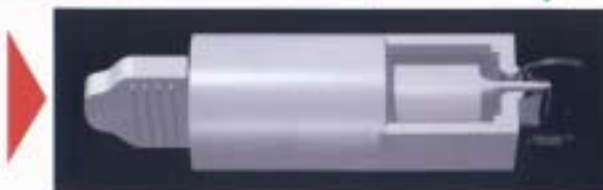

6. Now pull the plunger all the way out, to release the folded lens.



7. Using your implantation forceps, grasp the folded lens.



8. Remove the lens from the Delivery System. You are now ready for implantation.