



Single piece hydrophilic acrylic Lenstec Softec HDY

- » Most accurate IOL with patented Bi-aspheric design with square edge technology
- » Greater precision due to quarter diopter increments
- » Tighter manufacturing diopter tolerance within +/- 0.11 D
- » Proven quality for the most accurate cataract surgery



more information on www.lenstec.com

Quarter diopter IOL | Bi-Aspheric Softec

Technical Specifications⁺

Optic Size	5.75 mm		
Optic Type	Equal Conic Bi-aspheric		
Length	12.00 mm		
Haptic Style	Modified C		
Angulation	0 Degrees		
Positioning Holes	0		
Construction	1 Piece		
Optic Material	Acrylic (26% Water Content)		

[†]A Constant and A/C Depth figures shown are strictly guidelines for the calculation of implant power. Lenstec recommends that surgeons develop their own values based on technique, measuring equipment, and desired postoperative results.

Constants (Optical Interferometry):*

Immersion	A = 118.3
SRK/T	A = 118.3
Holladay1	sf = 1.39
Hoffer Q	pACD = 5.14

i.e. Using IOL Master, LENSTAR Optical Biometers

Diopter Steps

Whole	+5.0 to +36.0		
Half	+10.5 to +29.5		
Quarter	+15.0 to +25.0		

Softec HDY - The Blue Blocking Lens

Single piece hydrophilic acrylic (26% Water Content)

The Softec HDY is fabricated from a material that has been proven stable, particularly with respect to long term exposure to sunlight, using an accelerated aging study that replicates the equivalent of 20 years of exposure for the average person (based on specific ISO standards for the photostability of IOLs). Using blue light absorbers in the fabrication of IOLs has been an ongoing discussion since the late 1980s. Most researchers would agree that blocking certain wavelengths of visible light is beneficial to the eye, and a material proven to provide stable light transmission properties for over 20 years is the obvious choice.


Softec HD New Gold Standard - 0.11 DTolerance - 3x More Precise! Optical Prescription Selection and Tolerance Example: eye requires 24.25D Prescription to achieve optimal vision.

Industry Standard IOL		Softec HD IOL			
Lens Presciption	Allowed Tolerance	Max. Variance for a standard	Lens Presciption	Allowed Tolerance	Max. Variance for a Softec HD
24.00	±0.4	IOL in an eye that	24.00	±0.11	IOL in an eye that
		requires a 24.25D	24.25	±0.11	requires a 24.25D
24.50	±0.4	lens is 0.65D .	24.50	±0.11	lens is 0.11D .
		(smaller number is better)	24.75	+0.11	(smaller number is better)

Softec HD is the only IOL designed to address both Spherical Aberration and Defocus. Defocus is a more significant aberration than Spherical Aberration.

Bi-Aspheric Equal Conic Zero aberration IOL. Softec HD addresses the issue of spherical aberration inherent in conventional

monofocal spherical IOLs by adjusting the optic with a patented design on both the anterior and posterior surfaces.

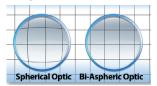

Studies have shown that Aspheric IOLs provide patients with significant optical benefits over traditional spherical surface IOLs . 1,3,4

Significant Outcomes. Mean refractive outcome was found to be closer to intended outcome, Depth of field was significantly improved, and Critical print size for fluent reading was smaller when compared to a standard monofocal IOL. ²

Proven quality

Stability of the biomaterial from which the intraocular Lenstec lenses are made, is long-term study proven and confirmed by millions of implanted lenses worldwide.

1.Thibos L, Hong X, Bradley A, Chang X. Statistical variation of aberration structure and image quality in a normal population of healthy eyes. J. Opt. Soc. Am A, Vol 19. No 21/Dec 2002 » 2. Craig J, Shah S, Wolffsohn J. Clinical evaluation of the Softec HD aberration-free aspheric intraocular lens. Submitted for publication. » 3. Sarver E. Theoretical optical performance of an equal conic intraocular lens and comparison to spheri cal and aspheric IOLs. AAO Presentation 2005 » 4. Nanavaty M, et al. Wavefront aberrations, depth of focus, and contrast sensitivity with aspheric and spherical intraocular lenses: fellow eye study. J Cataract Refract Surg. 2009; 35: 663 - 671

World Headquarters

Lenstec, Inc. 1765 Commerce Ave. N. St. Petersburg, FL 33716 USA

Tel: 727-571-2272 Fax: 727-571-1792 Email: lenstec@lenstec.com

Lenstec Barbados

Lenstec Barbados Inc. Airport Commercial Centre Pilgrim Road, Christ Church BARBADOS BB17092

Tel: 246-420-6795 Fax: 246-420-6797 Email: lenstecbarbados@lenstec.com

Lenstec UK

Lenstec Barbados Inc. Lenstec House Unit 8, Mariner Court Calder Park Wakefield WF4 3FL, UK

Tel: +44 (0)1924 382 678 Fax:+44 (0)1924 850 454 Email: lenstecuk@lenstec.com



