
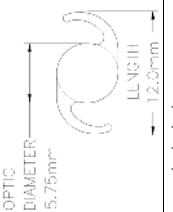
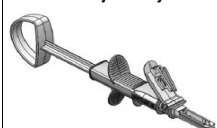


IMPLANTAČNOU KARTOU

- Všetky polia na ID karte musia byť vyplnené zdravotníckym úradom/poskytovateľom.
- Zdravotnícke zariadenie / poskytovateľ zdravotníckych služieb nesú zodpovednosť za prilepenie nálepky s vyleptaným logom „LENSTEC“ na zadnej (nepotačenej) strane ID karty pacienta a predanie tejto položky pacientovi ako záznamu o implantácii.

MODEL	POWERS (D)	TOLERANCES (D)	INCREMENT (D)	ACCESSORIES
 <p>OPTIC DIAMETER 5.75mm LENGTH 12.0mm</p> <p>SOFTEC HD™ Click</p>	+5.0 to +10.0	±0.25	1.0	<p>Click Injection System</p>  <p>Injector tip 2.0mm</p> <p>REF # 200-12-003-0010102</p>
	+10.5 to +14.5	±0.25	0.5	
	+15.0 to +25.0	±0.11	0.25	
	+25.5 to +30.0	±0.25	0.5	
	+31.0 to +36.0	±0.5	1.0	

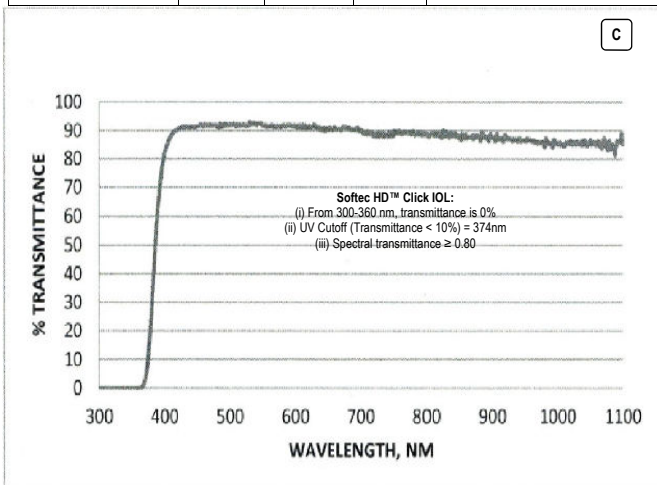

¹ Holladay, J. Quality of Vision: essential optics for the cataract and refractive surgeon. Pgs 27-35

² Pandey SK, Apple DJ, et al. Posterior Capsule Opacification: A Review of the Aetopathogenesis, Experimental and Clinical Studies and Factors for Prevention. Indian J Ophthalmol 2004;52:99-112

³ Apple DJ, Kleinmann G, et al. A new classification of calcification of intraocular lenses. Ophthalmology. Jan 2008, Volume 115, Issue 1, Pages 73-79



http://www.lenstec.com
ISO 13485 Registered Company

LENSTEC (BARBADOS) INC., Airport Commercial Centre, Pilgrim Road, Christ Church BB17092, Barbados
 +1 (246) 420-6795 +1 (246) 420-6797 lenstecbarbados@lenstec.com

CMI spol. s r.o. Trenčianska 47, 821 09 Bratislava, Slovakia
 +421 (2) 52631441 +421 (2) 52922882 lenstec@cmi.sk

UK SALES: LENSTEC (BARBADOS) INC., Lenstec House, Unit 8, Mariner Court, Calder Park, Wakefield, WF4 3FL, UK
 +44 (0) 1924 382 678 +44 (0) 1924 850 454 lenstecuk@lenstec.com

WORLD HEADQUARTERS: LENSTEC, INC., 1765 Commerce Ave N, Saint Petersburg, Florida 33716, U.S.A.
 +1 (727) 571-2272 +1 (727) 571-1792 lenstec@lenstec.com

SoftecHD™

Click

CE
2797

BASIC UDI
08443690SOFTECHDCLICK5S

PK133 Rev 2 CS/SK
27/09/2022



NOTE: This package only contains the intraocular lens

STERILE
Sterilized using steam

Do not reuse


Do not resterilize

Do not use if damaged


Caution. Consult accompanying documents

R_{only}


1 & 2



3 & 4




5




1st Click: The MATC has connected to the lens holder. The system is now ready to use.

6

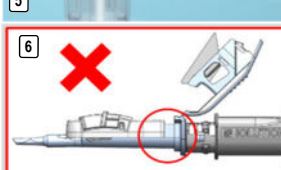


7

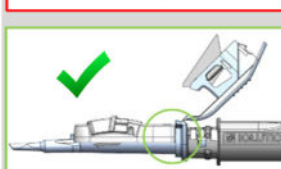


100% ready to use.


8

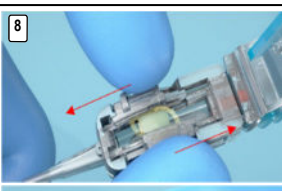

9




10



2nd Click: The IOL lens is visible.

11

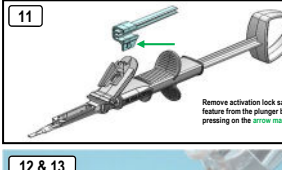

Remove activation lock safety feature from the plunger by pressing on the arrow mark.

12 & 13

