

LENSTEC (BARBADOS) INC., Airport Commercial Centre, Pilgrim Road, Christ Church BB17092, Barbados
 +1 (246) 420-6795 +1 (246) 420-6797 lenstecbarbados@lenstec.com


EC REP CMI spol. s.r.o. Trenčianska 47,821 09 Bratislava, Slovakia
 +421 (2) 52631441 +421 (2) 52922882 lenstec@cmi.sk

UK SALES: LENSTEC (BARBADOS) INC., Lenstec House, Unit 8, Mariner Court, Calder Park, Wakefield, WF4 3FL, UK
 +44 (0) 1924 382 678 +44 (0) 1924 850 454 lenstecuk@lenstec.com

WORLD HEADQUARTERS: LENSTEC, INC., 1765 Commerce Ave N, Saint Petersburg, Florida 33716, U.S.A.
 +1 (727) 571-2272 +1 (727) 571-1792 lenstec@lenstec.com

http://www.lenstec.com
 ISO 13485 Registered Company

Softec HD3

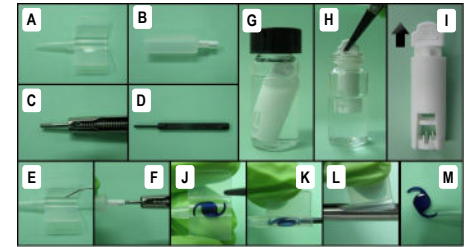

CE
2797



PKI43 Rev 1
TR/AR

DD-MM-YYYY

BASIC UDI

Softec HD3: 08443690SOFTECHD3MM

- 0
- Holladay, J. Quality of Vision: essential optics for the cataract and refractive surgeon. Pgs 27-35
 - Pandey SK, Apple DJ, et al. Posterior Capsule Opacification: A Review of the Aetiopathogenesis, Experimental and Clinical Studies and Factors for Prevention. Indian J Ophthalmol 2004;52:99-112
 - Apple DJ, Kleinmann G, et al. A new classification of calcification of intraocular lenses. Ophthalmology. Jan 2008, Volume 115, Issue 1, Pages 73-79



Sterilized using steam



Do not reuse



Do not resterilize



Do not use if damaged



Caution. Consult accompanying documents



Rx only

MODEL	POWERS (D)	TOLERANCES (D)	INCREMENT (D)	ACCESSORIES
 Softec HD3™	-5.0 to +10.0 +10.5 to +30.0 +31.0 to +36.0	±0.25 ±0.25 ±0.5	1.0 0.5 1.0	Use Implantation Forceps

